


**Bio SB**  
BIO SCIENCE FOR THE WORLD

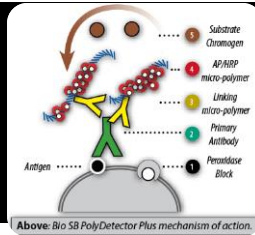

**IVD**

For In Vitro Diagnostic Use

## Mouse/Rabbit PolyDetector Plus AEC HRP Red Detection System

### Intended Use

For In Vitro Diagnostic Use

### Summary and Explanation

The **Mouse/Rabbit PolyDetector Plus AEC HRP Red System** is a 2-step polymeric detection system that allows for the demonstration of antigens in paraffin-embedded tissue, cryostat sections, blood smears, cytosmears, and cell preparations. The PolyDetector kits have been developed using a proprietary tandem hyperlabelling technology used to directly label immunoglobulins with enzymes. This ensures consistent and reproducible immunostaining for all types of nuclear, cytoplasmic and membranal antigens, in different types of tissues.

**Mouse/Rabbit PolyDetector Plus AEC HRP Red Detection System** is suitable for use with mouse IgG and IgM and rabbit primary antibodies, both monoclonal and polyclonal. The **Mouse/Rabbit Plus PolyDetector AEC HRP Red Detection System** kits are optimized for use with Bio SB primary antibodies; however, they are universal kits and therefore should work equally well with prediluted and concentrated antibodies from different vendors, as long as they are optimized.

Availability	
Catalog Number	Volume
BSB 0258S	5 mL Each
BSB 0258	15 mL Each
BSB 0260	50 mL Each
BSB 0262	100 mL Each
BSB 0264	200 mL Each
BSB 0266	1000 mL Each
BSB 0267	15 mL Plus Link & HRP Label
BSB 0268	50 mL Plus Link & HRP Label
BSB 0269	100 mL Plus Link & HRP Label
BSB 0270	200 mL Plus Link & HRP Label
BSB 0271	1000 mL Plus Link & HRP Label

### Presentation

The **Mouse/Rabbit PolyDetector Plus AEC HRP Red Detection System** contains a Peroxidase Bocker solution, A PolyDetector Plus Link solution, an Anti-Mouse/Rabbit Horseradish Peroxidase solution, and an AEC ready-to-use Substrate-chromogen solution. All the components are buffered with stabilizers and an anti-microbial.

**Storage:** Store at 2°C - 8°C

### Stability:

Stable up to the expiration date listed on the label. Do not use this product after the expiration date listed on the product label.

### Protocol

#### Preparation of Working Solutions

The **PolyDetector Peroxidase Blocker**, the **PolyDetector Plus Link** the **Anti-Mouse/Rabbit Horseradish Peroxidase Label**, and the **AEC Chromogen Solution** are ready-to-use working solutions and require no further preparation.

#### Mounting Protocol

##### A. AquaMounter Protocol

1. After the histological, immunohistochemical or in situ hybridization staining procedure is completed, rinse slides in deionized water. Do not incubate tissue or cell specimens in solvents such as alcohol, toluene, or xylene.
2. After counterstaining, apply 1-3 drops of AquaMounter (BSB 0090- BSB0093) to each slide making sure the specimen is covered.
3. Coverslip.
4. Observe under a light microscope.

##### B. ChromoProtector Permanent Mounting Protocol

1. After the histological, immunohistochemical or in situ hybridization staining procedure is completed, rinse slides in deionized water. Do not incubate tissue or cell specimens in solvents such as alcohol, toluene, or xylene.
2. Using a Coplin jar or a staining dish, immerse slides with tissues in ChromoProtector (BSB 0151 – BSB 0156) or lay wet slides horizontally and apply sufficient drops of ChromoProtector (BSB 0151 – BSB 0156) to completely cover the tissue. Carefully spread ChromoProtector if needed, but avoid contacting the tissue.
3. Incubate slides for ten minutes at 60 °C to allow ChromoProtector to penetrate tissues.
4. Remove excess ChromoProtector by placing slides vertically over an absorbent material and let excess drain off into absorbent material. Do not rinse slides.
5. Allow slides to COMPLETELY air dry.

- NOTE: The ChromoProtector will protect tissue from drying artifacts during the air-drying process.
- When slides are completely dried, they can be mounted using most standard mounting methods such as aqueous or permanent.
- Organic solvent permanent mounting medium such as PermaMouncer (Cat# BSB 0094-0097) or similar permanent mounting media, or the organic solvent-free XyGreen PermaMouncer (Cat # BSB 0169-0174), can be added directly to the slide until the tissue or cell specimen is covered.
- If the permanent mounting medium does not spread evenly on the dry slide, the slide can be dipped in toluene or xylene for 1 – 2 seconds to aid spreading of the mounting medium. Use a minimum amount of mounting medium so that slides dry rapidly.
- Apply coverslip and air dry before microscopic observation.

### Recommended Immunohistochemical Protocol

- Cut and mount 3-4 micron formalin-fixed paraffin-embedded tissues on positive charged slides.
- Air dry for 2 hours at 58°C.
- Deparaffinize, dehydrate and rehydrate tissues.
- Subject tissues to heat epitope retrieval using a suitable retrieval solution such as ImmunoDNA Retriever with Citrate (BSB 0020-BSB 0023) or EDTA (BSB 0030-BSB 0033).
- Wash with 5 changes of IHC Wash buffer.
- Place slides in **PolyDetector Peroxidase Blocker** for 5 min.
- Wash with 3 changes of IHC wash buffer.
- Cover tissue with the **Primary Antibody** following manufacturer's recommended protocol. If using concentrated antibodies, we suggest using our **ImmunoDetector Protein Blocker/Antibody Diluent** to dilute antibodies.
- Wash with 3 changes of IHC wash buffer.
- Cover tissue with **PolyDetector Plus Link**, incubate for 15 min.
- Wash with 3 changes of IHC wash buffer.
- Cover tissue with **PolyDetector HRP Label**, incubate for 15 min.
- Wash with 3 changes of IHC wash buffer.
- Cover tissue with **AEC Chromogen**, incubate for 10 minutes.
- Rinse with 3 changes of IHC was buffer.
- Counterstain.
- Coverslip with a water-based mounting medium such as the Bio SB Aqua Mounter (BSB 0090-0093).

### Abbreviated Immunohistochemical Protocol

Step	PolyDetector Plus AEC HRP Red
Peroxidase Blocker	5 minutes
Primary Antibody	30-60 minutes
Plus Link	15 minutes
HRP Label	15 minutes
AEC Chromogen	5-10 minutes
Counterstaining	Time Varies with counterstain


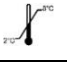





### Precautions

- For professional users only. Results should be interpreted by a medical professional.
- Ensure proper handling procedures are used with reagent. Always wear proper personal protective equipment such as laboratory coat, goggles and gloves when handling reagents.
- This product contains components with sodium azide (NaN<sub>3</sub>), a toxic chemical. At product concentrations it is not classified as hazardous due to its low concentration. Sodium azide may react with plumbing to form highly explosive build-ups of metal azides. Upon disposal, flush with large volumes of water to prevent metal azide build up.
- Minimize microbial contamination of reagents.
- Dispose of unused solution according to local and federal regulations.
- Do not ingest reagent. If reagent ingested, seek medical advice immediately.
- Avoid contact with eyes. Flush with large quantities of water if contact occurs.
- Follow safety precautions for the heating device (TintoRetriever Pressure Cooker or similar).
- For complete recommendations for handling biological specimens please refer to the CDC document, "Guidelines for Safe Work Practices in Human and Animal Medical Diagnostic Laboratories" (1).

### References

- U.S. Department of Health and Human Services: Centers for Disease Control and Prevention. Guidelines for Safe Work Practices in Human and Animal Medical Diagnostic Laboratories. Supplement / Vol. 61, January 6, 2012.

### Symbol Key / Légende des symboles/Erläuterung der Symbole

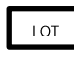	In Vitro Diagnostic Medical Device Dispositif médical de diagnostic in vitro In-Vitro-Diagnostikum		Storage Temperature Limites de température Zulässiger Temperaturbereich		Manufacturer Fabricant Hersteller		Catalog Number Référence du catalogue Bestellnummer
			Read Instructions for Use Consulter les instructions d'utilisation Gebrauchsanweisung beachten		Expiration Date Utiliser jusque Verwendbar bis		Lot Number Code du lot Chargenbezeichnung



Bio SB, Inc. • 69 Santa Felicia Dr., Santa Barbara, CA 93117, USA  
Tel. (805) 692-2768 • Tel. (800) 561-1145 • Fax. (805) 692-2769  
• E-mail: [info@biosb.com](mailto:info@biosb.com) • Website: [www.biosb.com](http://www.biosb.com)