


CD34

Clone: QBEnd-10
Mouse Monoclonal

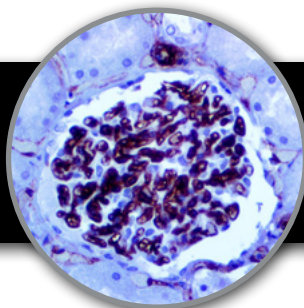

Bio SB
 BIOSCIENCE FOR THE WORLD

www.biosb.com

Inset: IHC of CD34 on a FFPE Kidney Tissue

Intended Use

For In Vitro Diagnostic Use.

This antibody is intended for use in Immunohistochemical applications on formalin-fixed paraffin-embedded tissues (FFPE), frozen tissue sections and cell preparations. Interpretation of results should be performed by a qualified medical professional.

Immunogen

Human endothelial vesicles.

Summary and Explanation

CD34 functions as a cell-cell adhesion factor and cell-surface glycoprotein. It may also mediate the attachment of stem cells to bone marrow extracellular matrixes or directly to stromal cells. Cells expressing CD34 are normally found in the umbilical cord and bone marrow as hematopoietic cells, and in vascular endothelium. In addition to stem cell recognition, CD34 is expressed by vascular endothelium; it appears that proliferating endothelial cells express this molecule in greater amounts than resting cells. In comparison to factor VIII R Antigen, CD34 stains are stronger and appear to be more sensitive in nature.

In tumors, CD34 is found in Alveolar Soft Part Sarcoma, pre B-ALL (positive in 75%), AML(40%), AMLM7 (most), Dermatofibrosarcoma Protuberans, Gastrointestinal Stromal Tumors, Giant Cell Fibroblastoma, Granulocytic Sarcoma, Kaposi's Sarcoma, Liposarcoma, Malignant Fibrous Histiocytoma, Malignant Peripheral Nerve Sheath tumors, Meningeal Hemangiopericytomas, Meningiomas, Neurofibromas, Schwannomas, and Papillary Thyroid Carcinoma. A negative CD34 may exclude Ewing's Sarcoma/PNET, Myofibrosarcoma of the breast, and Inflammatory Myofibroblastic tumors of the stomach.

Antibody Type	Mouse Monoclonal	Clone	QBEnd/10
Isotype	IgG1	Reactivity	Paraffin, Frozen
Localization	Cytoplasmic, Membranous	Control	Tonsil, Placenta, Appendix
Species Reactivity	Human, Dog		

Presentation

CD34 is a mouse monoclonal antibody derived from cell culture supernatant that is concentrated, dialyzed, filter sterilized and diluted in buffer pH 7.5, containing BSA and sodium azide as a preservative.

Presentations

Catalog Num.	Antibody Type	Dilution	Volume/Qty
BSB 5225	Tinto Prediluted	Ready-to-Use	3.0 mL
BSB 5226	Tinto Prediluted	Ready-to-Use	7.0 mL
BSB 5227	Tinto Prediluted	Ready-to-Use	15.0 mL
BSB 5228	Concentrated	1:100 - 1:500	0.1 mL
BSB 5229	Concentrated	1:100 - 1:500	0.5 mL
BSB 5230	Concentrated	1:100 - 1:500	1.0 mL
BSB 5231	Control Slides	Not Applicable	5 slides

Precautions

1. For professional users only. Ensure results are interpreted by a medical professional.
2. This product contains sodium azide (NaN₃), a toxic chemical which may react with plumbing to form highly explosive build-ups of metal azides. Upon disposal, flush with large volumes of water to prevent sodium azide build-up.
3. Ensure proper handling procedures are used with reagent. Always wear proper laboratory equipment such as laboratory coat and gloves when handling reagents.
4. Unused solution should be disposed of according to local and federal regulations.
5. Do not ingest reagent. If reagent ingested, contact a poison control center immediately.
6. For complete recommendations for handling biological specimens please refer to the CDC document, "Guidelines for Safe Work Practices in Human and Animal Medical Diagnostic Laboratories" (5).

Storage

Store at 2-8 °C. Do not use after expiration date listed on package label. Temperature fluctuations should be avoided. Store appropriately when not in use, and avoid prolonged exposure to room temperature conditions.

Specimen Preparation

Paraffin sections: The antibody can be used on formalin-fixed paraffin-embedded (FFPE) tissue sections. Ensure tissue undergoes appropriate fixation to ensure best results. Pre-treatment of tissues with heat-induced epitope retrieval (HIER) is recommended using Bio SB ImmunoDNA Retriever with Citrate (BSB 0020-BSB 0023), ImmunoDNA Retriever with EDTA (BSB 0030-BSB 0033) or ImmunoDNA Digestor (BSB 0108-0112). See reverse side for complete protocol. Tissue should remain hydrated via use of Bio SB Immuno/DNA Washer solutions (BSB 0029 & BSB 0042).

Frozen sections and cell preparations: The antibody can be used for labeling acetone-fixed frozen sections and acetone-fixed cell preparations.

Staining Procedure

1. Cut and mount 3-5 micron formalin-fixed paraffin-embedded tissues on positive charged slides such as Bio SB Hydrophilic Plus Slides (BSB 7028).
2. Air dry for 2 hours at 58° C.
3. Deparaffinize, dehydrate and rehydrate tissues.
4. Subject tissues to heat epitope retrieval using a suitable retrieval solution such as ImmunoDNA Retriever with Citrate (BSB 0020-BSB 0023) or EDTA (BSB 0030-BSB 0033).
5. Any of three heating methods may be used:

a. TintoRetriever Pressure Cooker or Equivalent

Place tissues/slides in a staining dish or coplin jar containing the ImmunoDNA Retriever with Citrate or EDTA, and place in the pressure cooker. Add 1-2 inches of distilled water to the pressure cooker and turn heat to high. Incubate for 15 minutes. Open and immediately transfer slides to room temperature.

b. TintoRetriever PT Module or Water Bath Method

Place tissues/slides in a pre-warmed staining dish or coplin jar containing the ImmunoDNA Retriever with Citrate or EDTA at 95°-99° C. Incubate for 30-60 minutes.

c. Conventional Steamer Method

Place tissues/slides in a pre-warmed staining dish or coplin jar containing the ImmunoDNA Retriever with Citrate or EDTA in a Steamer, cover and steam for 30-60 minutes.

6. After heat treatment, transfer slides in ImmunoDNA Retriever with Citrate or EDTA to room temperature and let stand for 15-20 minutes.
7. For manual staining, perform antibody incubation at ambient temperature. For automated staining methods, perform antibody incubation according to instrument manufacturer's instructions.
8. Wash slides with IHC wash buffer or DI water.
9. Continue IHC staining protocol.

Recommended IHC Protocol

Step	ImmunoDetector AP/HRP	PolyDetector AP/HRP	PolyDetector Plus HRP
Peroxidase/AP Blocker	5 min.	5 min.	5 min
Primary Antibody	30-60 min.	30-60 min.	30-60 min.
1st Step Detection	10 min.	30-45 min.	15 min.
2nd Step Detection	10 min.	Not Applicable	15 min.
Substrate-Chromogen	5-10 min.	5-10 min.	5-10 min.
Counterstain	Varies	Varies	Varies

Performance Characteristics

Normal Tissues

Positive (+)	
capillaries of most tissues	umbilical artery
Vein (to a lesser extent)	
Negative (-)	
endothelium of most large Vessels	
endothelium of placental sinuses	
Abnormal Tissues	
Positive (+)	
benign tumours of blood vascular origin 22/22	
Lymphangiomas (weak focal reaction) 5/8	
Tumour cells in angiosarcoma (vasoformative areas) 17/23	
Tumour cells in angiosarcoma (solid areas) 13/24	
Proliferating vessels 40/40	
spindle cells in Kaposi's sarcoma 40/40	
Carcinoma 1/54	
spindle cell tumours 17/45	


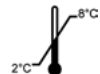






Product Limitations

Due to inherent variability present in immunohistochemical procedures (including fixation time of tissues, dilution factor of antibody, retrieval method utilized and incubation time), optimal performance should be established through the use of positive and negative controls. Results should be interpreted by a medical professional.

References

1. Civin CL, et al. London Academic Press. 1989:818-825
2. Fina L, et al. Blood. 1990;75:2417-2426
3. Ramani P, et al. Histopathology. 1990;17:237-242
4. Aziza J, et al. Am J Clin Pathol. 1990;96:25-31
5. U.S. Department of Health and Human Services: Centers for Disease Control and Prevention. Guidelines for Safe Work Practices in Human and Animal Medical Diagnostic Laboratories. Supplement / Vol. 61, January 6, 2012.

Symbol Key / Légende des symboles/Erläuterung der Symbole

	EMERGO EUROPE Prinsessegracht 20 2514 AP The Hague The Netherlands		Storage Temperature Limites de température Zulässiger Temperaturbereich		Manufacturer Fabricant Hersteller		Catalog Number Référence du catalogue Bestellnummer
	In Vitro Diagnostic Medical Device Dispositif médical de diagnostic in vitro In-Vitro-Diagnostikum		Read Instructions for Use Consulter les instructions d'utilisation Gebrauchsanweisung beachten		Expiration Date Utiliser jusque Verwendbar bis		Lot Number Code du lot Chargenbezeichnung

Bio SB
BIOSCIENCE FOR THE WORLD

69 Santa Felicia Dr., Santa Barbara, CA 93117, USA
Tel: (805) 692-2768 | Tel: (800) 561-1145 | Fax: (805) 692-2769
E-mail: info@biosb.com | Website: www.biosb.com

